

# KANGJI

## DISPOSABLE BALLOON UTERINE MANIPULATOR

Unparalleled Exposure  
Exceptional Control



## Easy to Operate

"S"-shaped integrated design conforms to the human body structure, making it easy and convenient to operate

## Position Lock (with magnifier)

The locking design for fast, accurate and secure positioning

## Shaft with Numbers

Dual scale markings on the shaft provide more precise operation

## Adjustable Cervical Cup

The cervical cup can be adjusted according to the length of the vagina to meet clinical requirements

## Integrated Design

The cervical cup and operating rod are integrated, which reduces the assembly

## Vaginal Balloon


The vagina position adopts balloon occlusion to prevent the leakage of CO<sub>2</sub>

## Groove Structure

The cervical cup mouth features a "V"-shaped groove structure, making it convenient for surgeons to position it during the procedure

## Uterine Balloon

Uterine balloon improves a better control in operation, suitable for different sizes of cavities

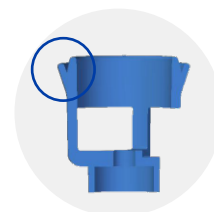


# Applications

TLH / LSH / LAVH

## Operation Procedure

- Sound the uterus and utilize cervical sizer to choose the most suitable cup size.
- Remove the fully-assembled instrument from its sterile packaging.
- Adjust the position of the cervical cup to align with the sounded number marked on the handle.
- Press the position lock to firmly affix the cup and start the procedure.

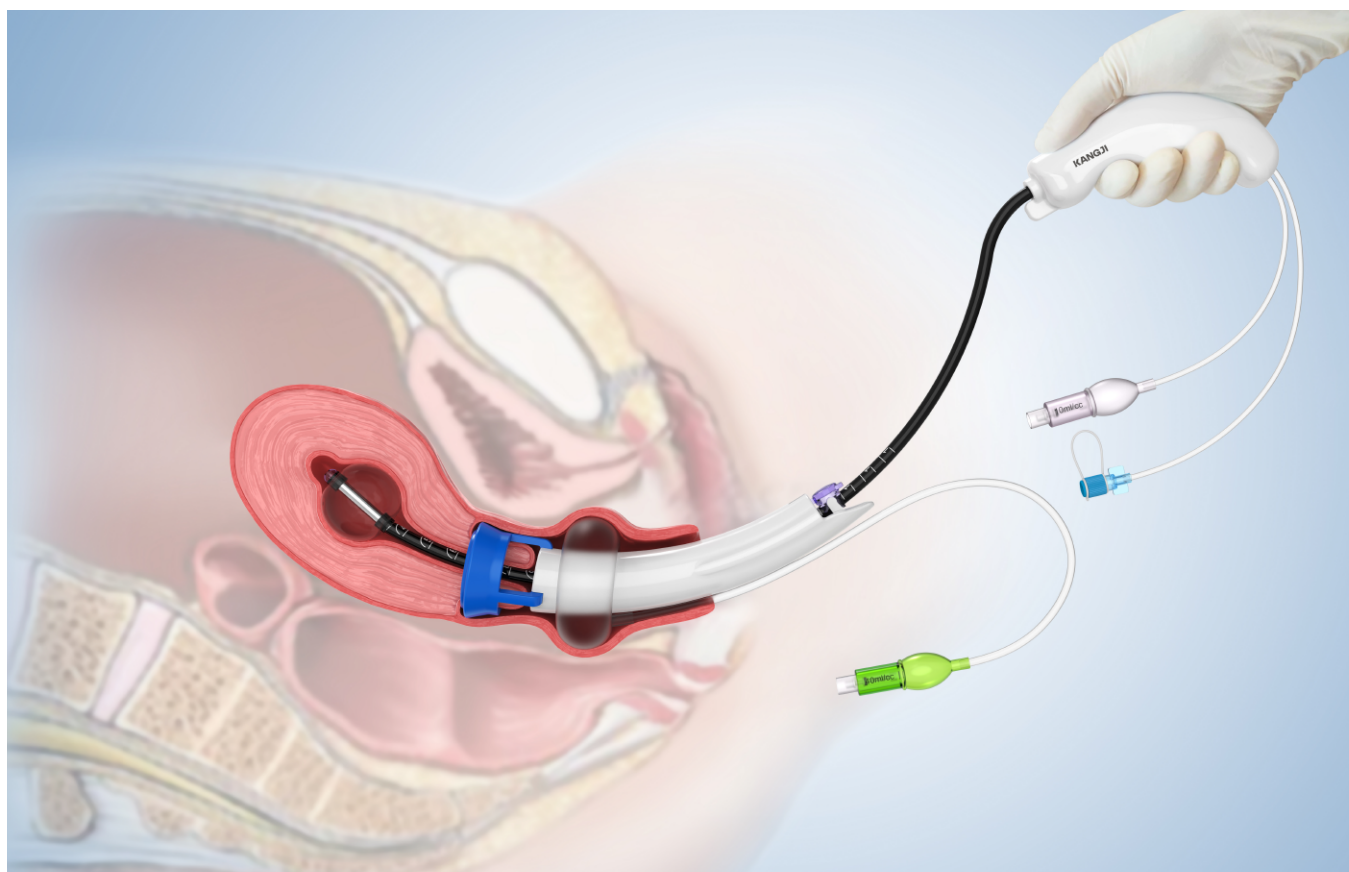

## Product Specification

Product	Code	OD of Cup	OD of Shaft
Disposable Balloon Uterine Manipulator	301Y.402	35 MM	6.0 MM
	301Y.403	38 MM	6.0 MM
	301Y.404	41 MM	6.0 MM

# KANGJI

**Hangzhou Kangji Medical Instrument Co., Ltd.**

Stock Code: 9997.HK

**Address** | No.1668 Chunjiang East Rd. Economic Development Zone,  
Tonglu, Hangzhou, Zhejiang 311501, China.

**Tel.** | +86-571-69900029

**Email** | [inquiry@kangji.com](mailto:inquiry@kangji.com)

**Website** | <https://www.kangjimed.com>    <https://www.kangjimd.com>



Follow us for more